**1. Intended use / Indication**

**RPR-1** is a radiation-curing, stable gel for the repair and modification of earmolds. The fine dispensing tips guarantee you optimum control and dosage for repair and venting work in the laboratory.

**2. Contraindication**

**RPR-1** is contraindicated ...

- ... if a patient is known to be allergic to any of the ingredients.
- ... for every application that is not part of the indication (see above).

**3. Patient Target Group**

Individuals receiving hearing aid treatment.

**4. Intended Users**

ENT physician, hearing aid acoustician

**5. Requirements**

**Hardware (Post Curing):**

pro3dure medical (CD-1, CD-2) · Am Burgberg 13, D-58642 Iserlohn, Germany

**6. Material**

**RPR-1** consists of functional (meth)acrylic resins, initiators, dyes and stabilisers.

**7. Geometric presetting**

Not applicable.

**8. Material parameters**

Not applicable.

**9. Manufacturing process (fig. 1-6)**

- Roughen the surface to be repaired.
- Degrease the areas to be roughened with a suitable agent (e.g. isopropyl alcohol).
- Apply **RPR-1** gel in the areas to be repaired.
- If necessary, fix with a UV polymerization lamp (handpiece).
- Curing of the **RPR-1** gel with suitable post-curing equipment (2 min. **CD-2**, under nitrogen).
- Post-processing and finishing of the repair area.

**10. Post-processing and finishing of the repair area**

When finishing the repaired areas, use the usual techniques and work steps (sanding/vibratory grinding). Please observe the instructions and notes of the relevant suppliers. Surface sealing can be done by polishing or painting (**L-1**).

**11. Notice**

Please follow the instructions provided by the hardware manufacturer in terms of parameter settings / post curing recommendations. To avoid detrimental effects on material quality do not expose the liquid material to irradiation under any circumstances. Deviations from the described manufacturing processes or storing conditions may lead to different mechanical and optical properties of the material. Ensure personal protective gear during processing.


**Caution:** Obtain special instructions before use. Do not handle until all safety precautions have been read and understood. Wear protective gloves / protective clothing / eye protection / face protection. If exposed: Call a POISON CENTER or doctor/physician. Store locked up. Dispose of contents/container in accordance with official regulation.

**12. Hazard statements**

May damage the unborn child. Suspected of damaging fertility.

**Repair gel · Reparaturgel  
Gel de réparation · Gel de reparación  
Gel di riparazione  
RPR-1**

Instructions for use · Gebrauchsanweisung  
Mode d'emploi · Instrucciones de uso  
Istruzioni per l'uso

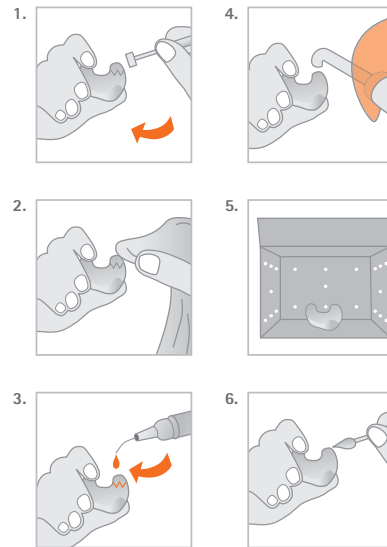Physical properties\* /  
Physikalische Eigenschaften\* /  
Propriétés physiques\* /  
Propiedades físicas\* /  
Proprietà fisiche\*:

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